INJECTED VISCOELASTIC SQUARE CUSHION EV805

Purpose:

Sitting cushion for prevention of sores. Risk - High

Features:

Injected Memory foam 80Kg/m³

Cover: PU fabric, anti-slip base, waterproof, zipper
Front with handle / back with buckles and webbing.

Size: 42 x 42 x 8 cm

Batch: See package

Color: Blue

Date of manufacture: See package

Model: EV805

Brand: GreenVisco

Maintenance and storage:

All cushions come with a removable cover, which allows you to wash them. It can be hand washed in cold water, using non-abrasive products or machine washed, using a programme for delicate clothes, cold and without centrifuges. It is recommended to use a safety net to avoid deformation of the cover during washing. Dry in the horizontal position;
Do not bleach; Do not iron.

Cushion core: Block with viscoelastic foam. -Do not wet.

Store the cushion preferably horizontally, avoiding sun exposure and moisture.

Instructions for use:

The antidecubitus cushion prevent the appearance of pressure ulcers, consequence from the permanence of the body in the same position for long periods of time. Pressure ulcers appear mainly in areas of bony prominence and support points of the body, such as heels, gluteus, shoulders, knees, elbows and pelvis. Excessive pressure on a point of the body, limits the intake of blood to that area and consequently oxygen, causing redness, destruction of the dermis/epidermis and consequently superficial or deep wounds.

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